


Easy to use... Always ready

Surgical Clipper Setup Guide*



Attaching

Place blade on top of clipper head. Slide blade back until indentations on clipper's side are aligned.

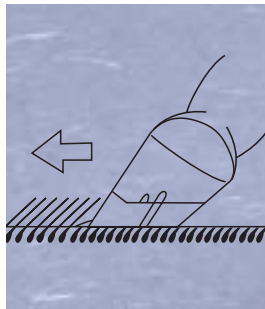

Clipping

With blade at 60° angle to skin, glide blade against surface of skin. For best results, clip against lie of hair. For difficult areas, stretch skin taut.



Removing

With blade facing downward, use thumb to push blade forward. Allow used blade to fall into appropriate receptacle.



Cleaning

Rinse clipper under warm or cold running water or submerge in water. Dry surface with clean towel.



Disinfecting

Wipe with surface disinfectant or soak in antimicrobial agent. Then, rinse with water and towel dry.




Charging

Switch clipper to off position. Place clipper handle in charging adapter. (Full charge requires 12 hours.)



Storing

Store clipper in upright position utilizing charging adapter 4414.

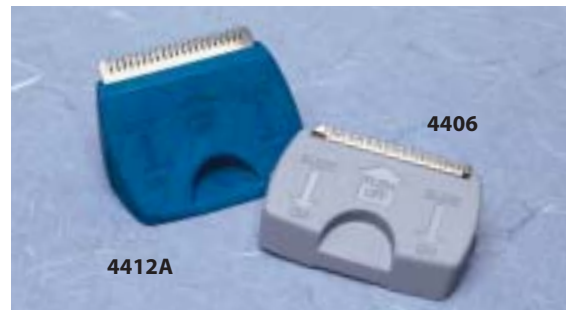

Surgical clipper blade options

Cat. No.	Description	Qty. Cs
4406	General purpose – for body hair	50
4412A	Specialty – for scalp and other thick, coarse hair	20



Single-use blades are easy to use and replace

Cat. No.	Description	Qty. Cs
4413	Surgical clipper	1
4414	Surgical clipper charging adapter	1
4406	General-purpose clipper blade	50
4412A	Specialty clipper blade	20



For more information, please contact your Cardinal Health, Medical Products and Services sales representative or customer service at 1.800.964.5227.

© Copyright 2003. Cardinal Health, Inc. or one of its subsidiaries. All rights reserved.
Lit. No. PSC00210A (0403/2.5M/MC1469)

♻️ Printed on recycled paper with 10% post-consumer fiber.



Cardinal Health
Prep Products
1500 Waukegan Road
McGaw Park, IL 60085

www.cardinal.com