


Convertors®

Sharps Containment and Instrument Management Blade Disarming Instructions

The Intranetics™ system of core surgical solutions from Allegiance Healthcare helps make fundamental OR safety more efficient regardless of usage protocol or surgical procedure. Whether your choice is


Custom Sterile™ packs, Convertors® standard packs or single sterile components, you can trust Allegiance to bring you products and services to make your job of delivering quality patient care easier.

Step One

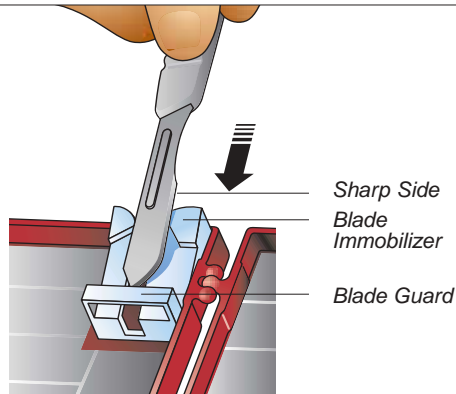


- > Turn the scalpel so that the blade edge faces the needle counter box hinges.
- > Push the blade handle forward above the BLADE IMMOBILIZER and below the BLADE GUARD rail.
- > Apply slight forward and downward pressure on the scalpel handle to seat the blade base into the BLADE IMMOBILIZER shoulders.

Step Two

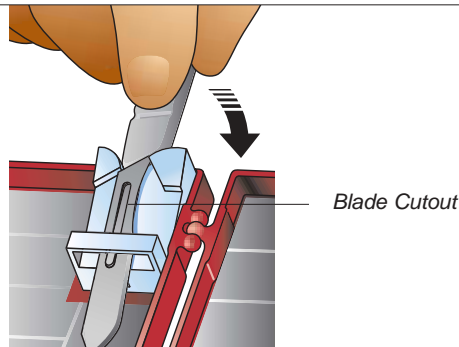

- > A slight click can be heard once the blade base seats into the BLADE IMMOBILIZER shoulders.
- > The scalpel blade should now be immobilized.
- > Apply a downward force on the scalpel handle to disengage the handle KEY SLOT from the base edge of the blade CUTOUT.

Step Three

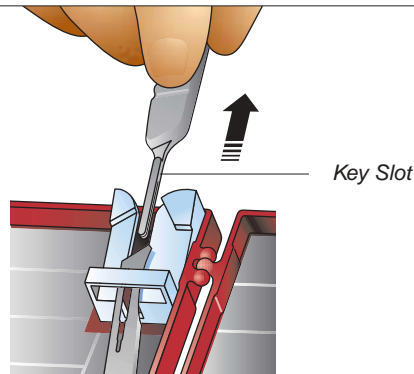


- > Remove the scalpel handle from the blade by pulling back from the blade disarmer until the handle KEY SLOT is fully removed.
- > The blade should tip forward onto the magnetic pad.
- > The scalpel handle should now be used to direct the removed blade to its final position.

For additional information, please contact your Allegiance Healthcare Sales Representative or Customer Service at 1.800.964.5227.

 **Allegiance**

a Cardinal Health company

© Copyright 2002, Allegiance Healthcare Corporation.
All rights reserved.
Lit. No. SS000190 (0402/2.5MMC1242)

 Printed on recycled paper with 10% post-consumer fiber.

Allegiance Healthcare Corporation
Convertors
1500 Waukegan Road
McGaw Park, IL 60085
www.cardinal.com/allegiance