


Synthetic Exam Glove Selection Guide

Glove Products

Procedural Recommendations

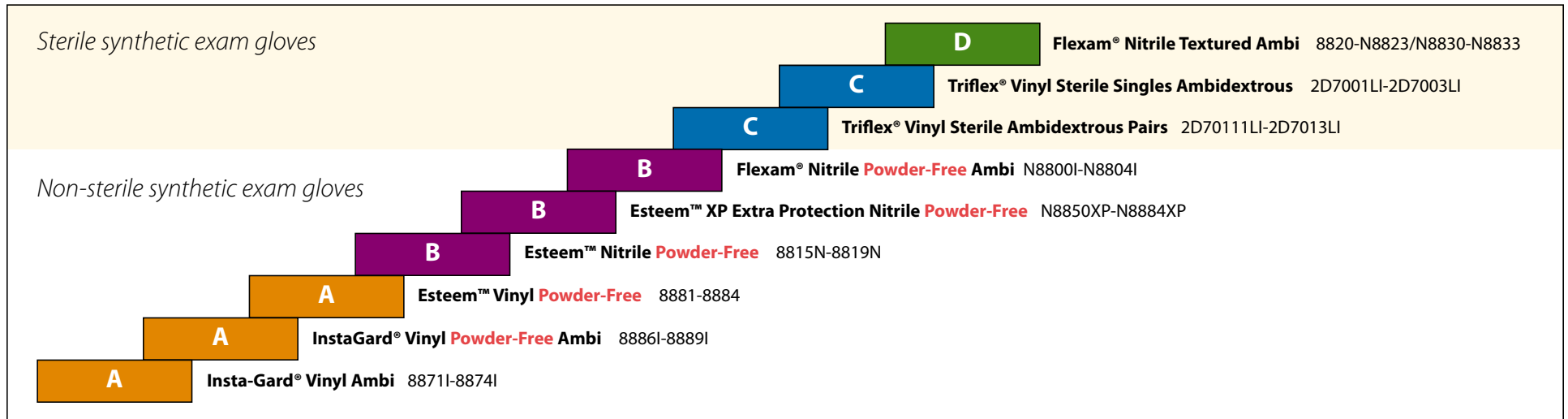

A Non-Sterile Vinyl

Low Risk of Exposure

- Changing bed linen
- Administering an injection
- Starting, D/C IV
- Brief suctioning
- Emptying emesis basin
- Managing urinary drainage bag
- Patient care <15 minutes

B Non-Sterile Nitrile

Moderate to High Risk of Exposure

- Preparation and administration of chemotherapy
- Direct patient care in ICU, ER, LDRP, OR
- Handling of laboratory specimens
- Patient care >15 minutes

C Sterile Vinyl

Low Risk of Exposure To Potentially Infectious Material

- Dressing change not saturated with blood or body fluid
- Minor suture removal
- Clinical intervention <15 minutes
- Low-risk patient-care units: Ophthalmology, Pediatrics, Non-Surgical Clinics, Employee Health Services

D Sterile Nitrile


Moderate to High Risk of Exposure to Potentially Infectious Material

- Dressing change
- Suture application/removal
- Vaginal or rectal exams
- Any clinical intervention < 30-minute duration that requires a sterile hand covering

Synthetic Surgical Glove Selection Guide

Glove Products

All gloves are sterile and packaged in surgical pairs

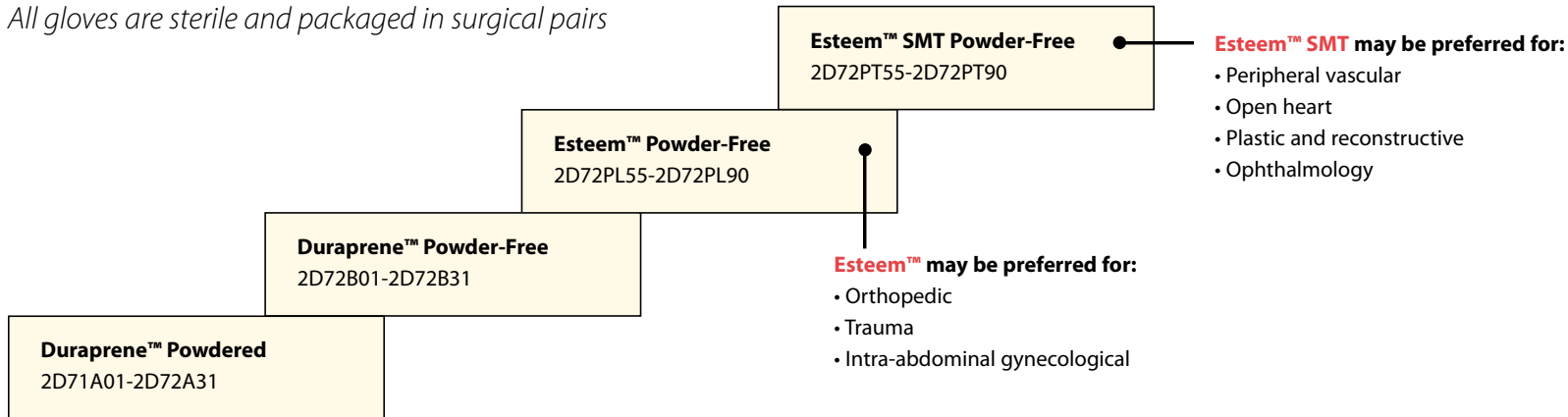

Clinical Recommendations:

All of these synthetic options may be used when a natural rubber latex glove is contraindicated or not preferred.